

The new  
**Pressure Infusion Cuffs**  
 for 500/1000ml und 1500ml bags  
 in pocket version



CE


**PRESSURE INFUSION CUFF 500/1000ml, universal,**  
 for all 500/1000ml bags, with female Luer-Lock Connector  
 dto., but with Hand Inflator  
 dto., but with Hand Inflator and Manometer, fixed to the Infusion Cuff  
 dto., but with Hand Inflator and Manometer

**PRESSURE INFUSION CUFF 1500ml**  
 for all 1500ml bags, with female Luer-Lock Connector  
 dto., but with Hand Inflator  
 dto., but with Hand Inflator and Manometer, fixed to the Infusion Cuff  
 dto., but with Hand Inflator and Manometer


Ref. No.  
 52-01-100  
 52-05-100  
 52-08-100  
 52-09-100

52-01-150  
 52-05-150  
 52-08-150  
 52-09-150

Instructions of use:

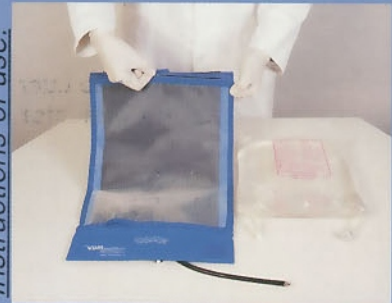

1. Open the zip-fastener.

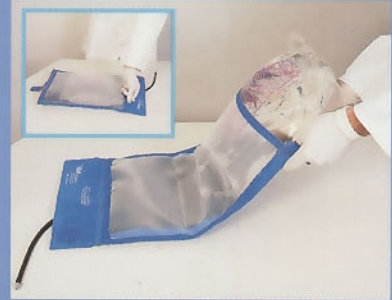

2. Put the bag inside the cuff  
 and pull the outlet of the bag  
 through the opening of the cuff.



3. Close the zip-fastener.



4. Connect the cuff to the  
 Automatic Pressure Infusor  
 or to a manual Pump.

**VBM**